

gloveon

Celeste LC



Nitrile Exam Gloves Powder Free, Long Cuff

GloveOn® Celeste LC is a light weight long cuff glove that dons like a second skin, yet delivers high tensile strength and resistance to chemicals. GloveOn® Celeste LC nitrile exam gloves are ideal for delicate procedures and jobs that demand absolute tactile sensitivity.



GloveOn® Celeste LC

GloveOn® Celeste LC		
Length (mm)	≥ 280	
Thickness Measurements (mm)		
Palm (centre of Palm)	0.07 ± 0.02	
Finger (13mm ± 3mm from tip)	0.10 ± 0.02	
Physical Properties	Before Ageing	After Ageing
Tensile Strength (MPa)	≥ 18	≥ 16
Elongation (%)	≥ 500	≥ 400
Inspection Levels & AQL	Inspection Level	AQL
Watertightness	G1	1.5
Physical Dimensions	S2	4.0
Tensile Strength	S2	4.0
Visual Inspection (Major)	S4	2.5
Visual Inspection (Minor)	S4	4.0
Particulate Residue	N = 5	≤ 2mg/glove

REORDER CODE

NTR56XS	X-SMALL
NTR56SS	SMALL
NTR56MM	MEDIUM
NTR56LL	LARGE
NTR56XL	X-LARGE

FEATURES

- Fingertip textured
- Powder free
- Not made with natural rubber latex
- Chemo drugs tested
- Lab chemical tested
- Ambidextrous
- Long cuff
- Violet blue colour

PACKAGING

150 gloves per box
10 boxes per carton

REGULATORY COMPLIANCE

TGA - ARTG 164563, FDA 510(k), MDD 93/42/EEC, REACH, ROHS Directive 2002/95/EC, EU 10/2011, EC 1935/2004, PPE 89/686/EEC

STANDARDS

ASTM D6319, EN 455 part 1, 2, 3 & 4, EN 1186, EN 13130, CEN/TS 14234, EN 420, EN 374 part 1, 2 & 3, HACCP Certified

MANUFACTURING ACCREDITATIONS

ISO 9001:2015
ISO 13485:2003
EN ISO 13485:2012

Chemotherapy Drugs and Concentration (Tested for Resistance to Permeation by Chemotherapy Drugs as per ASTM D6978-05-Test Report PN 104145A)

Chemotherapy Drugs and Concentration (Tested for Resistance to Permeation by Chemotherapy Drugs as per ASTM D6978-05-Test Report PN 104145A)	Minimum Breakthrough Detection Time (minutes)
Carmustine (BCNU), 3.3mg/ml (3,300 ppm)	1.2 minutes
Cisplatin, 1.0mg/ml (1,000 ppm)	>240 minutes
Cyclophosphamide (Cytoxan), 20.0mg/ml (20,000 ppm)	>240 minutes
Dacarbazine (DTIC), 10.0mg/ml (10,000 ppm)	>240 minutes
Doxorubicin Hydrochloride, 2.0mg/ml (2,000 ppm)	>240 minutes
Etoposide (Toposar), 20.0mg/ml (20,000 ppm)	>240 minutes
Fluorouracil, 50.0mg/ml (50,000 ppm)	>240 minutes
Methotrexate, 25.0mg/ml (25,000 ppm)	>240 minutes
Mitomycin C, 0.5mg/ml (500 ppm)	>240 minutes
Paclitaxel (Taxol), 6.0mg/ml (6,000 ppm)	>240 minutes
Thiotepa, 10.0mg/ml (10,000 ppm)	30.9 minutes
Vincristine Sulfate, 1.0mg/ml (1,000 ppm)	>240 minutes

WARNING: Carmustine and Thiotepa, at the tested concentration, degraded Celeste LC nitrile glove at 1.2 minutes and 30.9 minutes, respectively. The safe use of gloves in chemotherapy treatment is solely the decision of clinicians authorised to make such decision.

A brand by



Mun Australia
Toll free: 1800 456 837 (GLOVES)
info.au@munglobal.com



Protection Always On